ARCIOLANE

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Fractionated silicone oil
for retinal surgery

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ENGLISH

Arciolane® • COMPOSITION : Arciolane® fractionated, purified, sterile and apyrogenic silicone oil for vitreo-retinal surgery. **PRESENTATION AND PACKAGING :** Arciolane® is a medical grade purified and fractionated silicone oil , designated for prolonged tamponade of the retina. The product is available in pouched vial and pouched syringe for viscosities 1300 or 5500 cSt. A set of patient labels is provided to allow the traceability. This product does not contain Latex nor preservatives. **INDICATIONS :** prolonged tamponade after surgical treatment for severe retinal detachment (RD) especially : RD with giant tears, RD with proliferative vitreoretinopathy, traumatic RD. **CONTRA-INDICATIONS :** known hypersensitivity to silicone. Arciolane® should not be used in pseudophakic patients who have been fitted with a silicone IOL. **WARNING AND PRECAUTIONS :** As with all surgical procedures, the posterior segment surgery presents risks which the surgeon must evaluate ARCADOPHTA is not liable for any post-operative complications which arise following the use of Arciolane®. Some of the potential complications of retinal detachment surgery or of silicone oil implantation are : per- and post-operative ocular hypotony or hypertony , post operative choroidal detachment, corneal epithelium edema, secondary retinal detachment, cystoid macular edema, infection (endophthalmy), glaucoma by angle closure, keratitis, cataract, late emulsification of the silicone oil. **PRECAUTIONS :** Do not re-sterilize, single use only. Do not use if the sterile pouch has been compromised. Do not use after expiration date. It is recommended that silicone oil be removed once satisfactory results have been achieved. Silicone oil has a limited ability to absorb light that could result in excessive retinal damage in case of photocoagulation. Silicone oil has a comparatively high refractive index that can result in a significant refractive error when calculating IOL power using standard formulas. **DIRECTIONS FOR USE :** Before use, check the integrity of the individual pouch preserving the sterility. Vials : Transfer in aseptic conditions the content of the vial into a sterile syringe. Syringe : Before use remove the white stopper from the top of the syringe by rotating it a quarter turn. The syringe is ready to be used. Connect the syringe in a silicone oil injection machine. Removal :When removing the silicone oil, make sure that there are no silicone blebs in the vitreal cavity. Once withdrawn from the patient , Arciolane® should be discarded as any human biological product.. Strictly reserved for surgeons trained in retinal surgery. **STERILIZATION METHOD :** dry heat and autoclave. **STORAGE CONDITIONS :** keep at room temperature in a dry, protected area. **PHYSICAL PROPERTIES :**

Parameters	ARCIOLANE 1300	ARCIOLANE 5500
Viscosity (25°C)	1300 cSt	5500 cSt
Refractive Index (20°C)	1.404	1.404
Transmission spectrum (200 à 1200 nm)		

Manufactured by ARCADOPHTA, 293 route de Seysses, 31100 Toulouse, France. Tel. : 33(0)5 61 40 48 32 - Fax : 33(0)5 61 40 84 66. E-mail : info@arcadophta.com

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